# EULATIN® NH Zäpfchen

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MED recycling 1141333115

# EULATIN® NH Suppositories

Medical device to relieve bowel movement when suffering from haemorrhoids

## INSTRUCTIONS FOR USE

### 1. Therapeutic purpose

*Eulatin® NH Suppositories* achieve a physical relief of bowel movement by lubricating the rectum, the anus and by protecting the imperforate anal mucosa in case of haemorrhoids. The product improves the sliding properties of stool and protects the mucosa from laceration and itching.

### 2. Therapeutic indications

*Eulatin® NH Suppositories* are ready to use for internal haemorrhoids in the end region of the rectum and of the anus for the physical protection of the imperforate anal mucosa in adults. The product is designed and suitable for the professional care of patients, the handicapped and the elderly, and for domestic use.

### 3. Undesirable effects

Undesirable effects are not known for this product when used as prescribed.

### 4. Incompatibility, Interactions

No incompatibility is known for this product. *Eulatin® NH Suppositories* should neither be applied in combination with other creams and ointments in the anal areas nor with medicaments used rectally.

### 5. Contraindications

*Eulatin® NH Suppositories* may not be used when allergies to one of the ingredients are known.

#### Attention:

Product may not be used during pregnancy or lactation.

### 6. General special precautions

The product should not be used if blood is discharged or blood is suspected in stool and if there is simultaneous occurrence of pain and fever.

The product is solely designed for application to the treatment area and may not enter the body by ingestion.

**Attention:**  
Keep this medical device out of the reach of children.  
Keep in a safe place!

**Attention:**  
After the usage of *Eulatin® NH Suppositories* the tensile strength of latex condoms may be reduced and thus lead to their reduced protection.

### 7. Product properties

Haemorrhoids cause discomfort which, depending on the severity of the disorder, may lead to itching, bleeding, inflammation or thrombosis. Severe pain is experienced particularly in bowel movement. As a reflex to the severe pain experienced in bowel movement, unconditioned constipation occurs, and more intensive pressing strain is required for bowel movement. This then leads to a further amplification of existing haemorrhoids resulting in even more pressing strain being required.

As the product melts at body temperature, *Eulatin® NH Suppositories* lubricate the anal passage and give protection against haemorrhoids. The stool can slide more easily through the pre-lubricated anal passage, bowel movement therefore requires little pressure and painful pressing can be avoided.

In addition, the anal mucosa is optimally protected against injury by the covering property of the product.

The gauze strip on *Eulatin® NH Suppositories* prevents the suppository from being pressed into rear bowel sections by the sphincter muscle. This strip retains the suppository in the lower end of the anal bowels and can thus achieve lubrication of the rectum and of the anal canal.

### 8. Product application

The anal areas as well as your hands should be cleaned carefully with tepid water before each application of *Eulatin® NH Suppositories*.

To withdraw the product remove the packaged suppository from the blister strip, pull the two blister halves at the tip of the suppository apart and withdraw the suppository with clean and uninjured fingers.

Insert *Eulatin® NH Suppositories*, twice a day, in the anus like conventional suppositories. The gauze strip of the suppository is being fixed by the sphincter muscle and remains outside of the anus. The gauze strip can be removed after about 2-3 hours retention time or with the next bowel movement.

The continued application should not exceed four weeks.  
As the product melts at body temperature, it is advisable to use something to protect your underwear, e.g. a panty liner.

Used containers should be disposed of separately in domestic waste, expired containers or residuals should be brought to chemist's for disposal.

### 9. Physical product data

#### Composition:

1 *Eulatin® NH Suppository* contains hard fat, alkaline bismuth gallate, tannic acid, highlydispersed silicon dioxide, an integrated gauze strip.

Appearance, odour: yellowish-brown suppository with affixed gauze strip, odourless.

Shelf-life: 2 years

The minimum shelf-life is printed on the blister and the outer package of the product.

Special precautions for storage:

Store *Eulatin® NH Suppositories* at room temperature but not above 25 °C.

### 10. Delivery specifications

The outer package of the product contains 2 blister strips with 5 individually packed suppositories à 2 g each.

### 11. Manufacturer / Distributor

Manufacturer and distributor of the medical device:

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RIEMSER Arzneimittel AG