# Dolo-Arthrosenex® M Salbe

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1142038/18

# Dolo-Arthrosenex® M Ointment

Medical device to support massage and for physiotherapeutic treatment

## INSTRUCTIONS FOR USE

### 1. Therapeutic purpose

*Dolo-Arthrosenex® M Ointment* is used as lubricant on uninjured skin to support massage in physiotherapeutic and prophylactic treatment of the muscular, tendon and joint system. The physical lubricant effect of the ointment supports the flowing and effective application of the kneading, rubbing and stroking grips. The lubricant, sliding and suppleness of the massage movements are improved deeper relaxation and loosening of the muscles is made possible.

### 2. Therapeutic indications

*Dolo-Arthrosenex® M Ointment* is used as aid for therapeutic massage on uninjured skin, applied regionally in the case of sports trauma, disorders of the musculoskeletal system such as arthrosis, hip pains, muscular pains, for the after-treatment of muscle injuries, rheumatic disorders, inter-vertebral disk pains, sciatica, muscle hardening, contusions, strains or sprains. The use of *Dolo-Arthrosenex® M Ointment* is particularly suitable also for prophylactic massages to prepare the muscles for heavy sportive strain and during and after sports to loosen, relieve and relax the muscles.

The product can be applied by adults in the domestic environment themselves and indirectly professionally to patients, the handicapped and the elderly.

The indirect application to children (elder than 6 years) is reserved to adults exclusively.

### 3. Undesirable effects, histocompatibility

The application of *Dolo-Arthrosenex® M Ointment* may, in rare cases, lead to hypersensitivity reactions. These are manifested by the appearance of local irritations, the formation of nodules and vesicles, usually accompanied by itching. In individual cases severe forms of skin reaction (Stevens-Johnson-Syndrome) may be experienced.

### 4. Contraindications

*Dolo-Arthrosenex® M Ointment* may not be applied when there is a hypersensitivity to one of the ingredients, particularly to one of the offsprings of salicylic acid.

The product may not be applied to inflamed, injured skin, eczema or psoriasis at place of application.

### 5. General special precautions

*Dolo-Arthrosenex® M Ointment* is solely designed for external application to injured skin and may not enter the body by ingestion, injection or infusion and may not come into contact with the eyes, mucous membranes or open wounds.

#### Attention:

*Dolo-Arthrosenex® M Ointment* may only be applied by adults. The application to nurslings and children under the age of 6 and to pregnant and breast-feeding women is excluded. Neither may *Dolo-Arthrosenex® M Ointment* be applied to patients suffering from disorders of the kidneys, the liver, the gastro-intestinal tract, the respiratory tract, from asthma or if they suffer from allergies.

#### Attention:

Wash hands thoroughly with water and soap after application.

#### Attention:

Check *Dolo-Arthrosenex® M Ointment* for integrity before taking it from the packaging.

#### Attention:

Keep out of reach and sight of children!

### 6. Product properties

*Dolo-Arthrosenex® M Ointment* makes the skin well-lubricated and pliable for physiotherapeutic massage. The physical sliding properties of the ointment support the flowing and effective kneading, rubbing and stroking grips. Improved sliding qualities and pliability allow a deep relaxation and loosening of the muscles.

Slight quantities of salicylic acid have been added to the product as preservation agent.

### 7. Product application

Massage is a widespread physical form of pain treatment which can, to some extent, be applied oneself. The varying techniques range between the application of considerable pressure and a gentle touch, between gentle stroking massaging, rubbing, tapping and vibrations. The classical massage is used to improve blood circulation, to loosen, stretch and relieve the muscles. The skin is supplied with blood, ligaments and tendons become more elastic (similar effect as warming up training), the limbs, or the joints respectively, are loosened and painful tension is relieved.

*Dolo-Arthrosenex® M Ointment* is used as required twice to three times a day, applied as a thin layer over the edge of the pain region and rubbed in.

It may be applied for a short time until the symptoms subside, however a maximum application period of 4 weeks may not be exceeded.

After drawing the product from the tube, close the tube tightly again and wash hands thoroughly after treatment!

The used up tube and folding box are disposed of in domestic waste, preferably separated.

### 8. Physical product data

#### Composition:

Purified water, Cetyl stearyl alcohol, Sodium cetyl stearyl sulphate, non-ionic emulsifier, medium chain triglycerides, Polyoxyethylene-12-Cetyl stearyl alcohol, Oleyl oleate, Propylene glycol, Menthol, Citrus oil, Perfume oil, Citric acid monohydrate, Alpha tocopherol, Palmitoylascorbic acid, Sodium edetate, (2-Hydroxyethyl) salicylate as preservation agent.

#### Appearance, odour:

white homogeneous ointment, smell at most only slightly perceivable

#### Storage conditions:

*Dolo-Arthrosenex® M Ointment* is stored at room temperature (not over 25 °C) under dry conditions, protect package from direct sun.

Close tube tightly again after use.

Shelf-life: 2 years

### 9. Delivery specifications

Tube with 100 g ointment, packed in folding box

### 10. Manufacturer / Distributor

Manufacturer and distributor of the medical device:

 Abanta Pharma GmbH  
Reichsstraße 78  
58840 Plettenberg  
Telefon: +49 341 2582 190  
Telefax: +49 341 2582 191  
E-Mail: info@abanta-pharma.de

#### Note:

Please read instructions for use.

EXP

Please read instructions for use

Manufacturer

LOT

Batch

Protect against direct sunlight

Temperature limitation

This leaflet was last approved in November 2017.

1142038/18

 Abanta  
Pharma